# Dino-Lite

Digital Microscope Medical

## Medical Manual

[www.dino-lite.eu](http://www.dino-lite.eu)

English .....	6
Français .....	11
Deutsch .....	20
Español .....	25
Italiano .....	34
Nederlands .....	39
Svenska .....	48
Polski .....	53
Dansk .....	62
Hrvatski .....	67
Português .....	76
Suomi .....	81

System requirements / Exigences du système /  
Systemanforderungen / Requisitos del sistema /  
Requisiti di sistema / Systeemvereisten /  
Wymagania systemowe / Systemkrav / Systemkrav

- Windows XP / 7 / 8 / 10 / 11, macOS 10.14 or above /ou plus / oder höher, o posterior / o superior / of hoger/ lub nows ze / eller högre/ eller over /lub wyższy / eller ovanför / eller over
- USB 2.0 port
- 4 GB RAM or above /ou plus / oder höher, o posterior / o superior / of hoger/ lub nows ze / eller högre/ eller over /lub wyższy / eller ovanför / eller over
- 150 MB free disk space / espace de disque libre / freier Festplattenspeicher / espacio libre en disco / spazio libero su disco / vrije harde schijfruimte/ wolnej przestrzeni na dysku / ledigt hårddiskutrymme / fri harddiskplads / wolne miejsce na dysku / ledigt diskutrymme / ledig diskplads
- Motion-JPEG (MJPEG) Codec: may be required / peut-être nécessaire / kann erforderlich sein / puede ser necesario / potrebbe essere necessario / kan noodzakelijk zijn/ moze być wymagane / kan krävas / kan kræve / moze być wymagane / kan krävas / kan være påkrævet /
- **English:** For WiFi operations, a MEDLW model is required, as well as a user device with WiFi (IEEE 802.11n or higher). **Français:** Pour les opérations WiFi, un modèle MEDLW est requis, ainsi qu'un appareil utilisateur avec WiFi (IEEE 802.11n ou supérieur). **Deutsch:** Für WiFi-Operationen ist ein MEDLW-Modell erforderlich, ebenso wie ein Benutzergerät mit WiFi (IEEE 802.11n oder höher). **Español:** Para operaciones WiFi se requiere un modelo MEDLW, así como un dispositivo de usuario con WiFi (IEEE 802.11n o superior). **Italiano:** Per le operazioni WiFi è richiesto un modello MEDLW, così come un dispositivo utente con WiFi (IEEE 802.11n o superiore). **Nederlands:** Voor WiFi-operaties is een MEDLW-model vereist, evenals een gebruikersapparaat met WiFi (IEEE 802.11n of hoger). **Svenska:** För WiFi-operationer krävs en MEDLW-modell samt en användarenhet med WiFi (IEEE 802.11n eller högre). **Polski:** Do operacji WiFi wymagany jest model MEDLW, oraz urządzenie użytkownika z WiFi (IEEE 802.11n lub nowsze). **Dansk:** For WiFi-operationer kræves en MEDLW-model samt en bruger-enhed med WiFi (IEEE 802.11n eller højere). **Hrvatski:** Za WiFi operacije potreban je model MEDLW, kao i uređaj korisnika s WiFi-jem (IEEE 802.11n ili više). **Português:** Para operações WiFi, é necessário um modelo MEDLW, assim como um dispositivo do usuário com WiFi (IEEE 802.11n ou superior). **Suomi:** WiFi-toimintoja varten tarvitaan MEDLW-malli sekä käyttäjälaitteessa on oltava WiFi (IEEE 802.11n tai uudempi).

Type	Name	Resolution	Magnification
MEDL7DW	DermaScope® Polarizer HR	5.0 MP	10-50x
MEDL4DW	DermaScope® Polarizer	1.3 MP	10-50x
MEDL7DM	DermaScope® Polarizer HR 200x	5.0 MP	~10-70x & 200x
MEDL4DM	DermaScope® Polarizer 200x	1.3 MP	~10-70x & 200x
MEDLW4DW	DermaScope® wireless	1.3 MP	10-50x
MEDL4E	EarScope Pro	1.3 MP	55-90x
MEDL4EP	EarScope Pneumatic	1.3 MP	55-90x
MEDL3E	EarScope Basic	VGA	20-30x
MEDLW4E	EarScope Pro wireless	1.3 MP	55-90x
MEDLW3E	EarScope Basic wireless	VGA	20-30x
MEDL4N Pro	CapillaryScope 200 Pro	1.3 MP	20x-200x
MEDL4N5 Pro	CapillaryScope 500 Pro	1.3 MP	~500x
MEDL4N5	CapillaryScope 500	1.3 MP	~500x
MEDLW-4N Pro	CapillaryScope Pro 200x Wireless	1.3 MP	20x-200x
MEDLW-4N5 Pro	CapillaryScope Pro 500x Wireless	1.3 MP	~500x
MEDL7HM	TrichoScope Polarizer HR	5.0 MP	~10-70x & 200x
MEDL4HM	TrichoScope Polarizer	1.3 MP	~10-70x & 200x
MEDL3H	TrichoScope Basic	0.3 MP	~10-70x & 200x
MEDLW-4HM	TrichoScope Wireless	1.3 MP	~10-70x & 200x
MEDL4R	IriScope	1.3 MP	10-20x
MEDLW-4R	IriScope wireless	1.3 MP	10-20x
MEDL4FW	PodoScope	1.3 MP	10-50x

Wifi adapter	LED's	Polarizer	Metal Housing	Specific accessories
no	8 white	yes	yes	open cap, long cap, closed
no	8 white	yes	no	open cap, long cap, closed
no	8 white	yes	yes	closed contact cap
no	8 white	yes	no	closed contact cap
yes	8 white	yes	no	open cap, long cap, closed
no	6 white	no	no	specula (3-4-5 mm)
no	6 white	no	no	specula (2,5-4 mm)
no	4 white	no	no	specula (3-4-5 mm)
yes	6 white	no	no	specula (3-4-5 mm)
yes	4 white	no	no	specula (3-4-5 mm)
no	8 white	yes	no	nailshaped cap
no	8 white	yes	no	nailshaped cap
no	8 white	no	no	
yes	8 white	yes	no	nailshaped cap
yes	8 white	yes	no	nailshaped cap
no	8 white	yes	yes	closed contact cap
no	8 white	yes	no	closed contact cap
no	8 white	no	no	
yes	8 white	yes	no	closed contact cap
no	2 white/2 yellow	no	no	
yes	2 white/2 yellow	no	no	
no	8 white	yes	no	open cap, long cap, closed

# ENGLISH

Thank you for purchasing a medical microscope camera. This manual describes the medical devices and its usage.

The medical devices can be used with the DinoCapture software for computers with a Windows operating system. The DinoXcope software is designed to work with macOS. Usage of these software options is explained in separate user manuals that can be found at [www.dino-lite.eu](http://www.dino-lite.eu).

For use with the wireless adapter, please refer to the separate manual.

## Important safety information

- Avoid touching the lens with finger to protect the product from electrostatic damage
- Do not drop
- Keep dry

Medical device Class 1 -  
According to Medical Devices Regulation (EU) 2017/745

Instructions for Use: P/N M1, rev. 6, 2024Q1

Consult Instructions before Use

## Intended Use

Please check the label on Your Dino-Lite to see what model you have and find the appropriate page about the intended use in this manual.

## Index

Product overview .....	07
Software installation .....	09
Cleaning and maintenance, warranty, support .....	10

## Package contents

1. Digital microscope
2. Protective pouch
3. Instructions for use
4. Model-specific accessories (see table)

***Any serious incident that has occurred in relation to the device should be reported to the manufacturer and to the competent authority of the EU Member State in which the user and/or patient is established.***

***In the event of the packaging being damaged, unintentionally opened before use or if the packaging is exposed to environmental conditions outside of those specified, contact the manufacturer or its representative.***

## DermaScope® models

### Intended Use

The DermaScope® is intended to make images of spots of the intact skin as part of a dermatological diagnosis.

### Hardware features

The wireless models do not have a microtouch and can be controlled via the software program on your connected device.

1. Your Dino-Lite has a microtouch function at the cable end of the device. Touching this sensor will capture the current image.
2. This model has a polarization function that can be controlled by turning the adjustable wheel at the cap.
3. This model has exchangeable front covers/caps.
4. This model has a Macro Zoom function: Working with 2 magnification rates while maintaining the same working distance: a larger field of view at a lower magnification and a zoom function for higher magnification.

## EarScope models

### Intended use

The EarScope is intended to make images of the inner ear as part of an ear diagnosis.

### Hardware features

The wireless models do not have a microtouch and can be controlled via the software program on your connected device.

1. Your Dino-Lite has a microtouch function at the cable end of the device. Touching this sensor will capture the current image. The wireless models do not have a microtouch and can be controlled via the software program on your connected device.
2. This model has exchangeable front speculas. The speculas are provided in 3 sizes to be used for different sizes of the ear canal. Additional speculas are available for purchase from your regular supplier or from your Medical reseller.
3. The pneumatic EarScope model has a bulb to blow air into the ear to check the mobility of the eardrum, which is sealed off by the speculum.

**Note: Be careful not to apply too much pressure when squeezing the bulb.**

## CapillaryScope models

### Intended use

The CapillaryScope is intended to make images of microcapillar vessels.

### Hardware features

The wireless models do not have a microtouch and can be controlled via the software program on your connected device.

1. Your Dino-Lite has a microtouch function at the cable end of the device. Touching this sensor will capture the current image. The wireless models do not have a microtouch and can be controlled via the software program on your connected device.
2. The front cap has a special form to allow you to put it directly onto the rim of a nail.

### How to use the CapillaryScope

1. To achieve the best results, it is advised to apply some vegetal oil (like peanut oil) to the area that is going to be examined (usually the ring finger).
2. Do not touch the oil surface before applying the CapillaryScope.
3. Put the CapillaryScope right on the nail edge with the oil.

4. Adjust the focus and move the CapillaryScope slowly to find the edge of the nail. Then move the CapillaryScope to find the first row of the capillary vessel beneath the nail edge.

## IriScope models

### Intended use

The IriScope is intended to make images of the eye for eye diagnosis.

### Hardware features

The wireless models do not have a microtouch and can be controlled via the software program on your connected device.

1. Your Dino-Lite has a microtouch function at the cable end of the device. Touching this sensor will capture the current image. The wireless models do not have a microtouch and can be controlled via the software program on your connected device.
2. The IriScope has two light colors, white and yellow. The light color can be chosen from the software, white light is more suitable for light iris colors and the yellow light for darker iris colors.

**Warning note: Do not shine onto subjects eye for more than 4 minutes per day.**

## TrichoScope models

### Intended use

The TrichoScope is intended to make images of hair and scalp as part of trichological diagnosis.

### Hardware features

The wireless models do not have a microtouch and can be controlled via the software program on your connected device.

1. Your Dino-Lite has a microtouch function at the cable end of the device. Touching this sensor will capture the current image. The wireless models do not have a microtouch and can be controlled via the software program on your connected device.
2. Some models (MEDL4HM and MEDL4HM4) have a polarization function that can be controlled by turning the adjustable cap.

## PodoScope models

### Intended use

The PodoScope is intended to make images of feet, toe nails and spots of the skin as part of a podiatric diagnosis.

### Hardware features

The wireless models do not have a microtouch and can be controlled via the software program on your connected device.

1. Your Dino-Lite has a microtouch function at the cable end of the device. Touching this sensor will capture the current image.
2. This model has a polarization function that can be controlled by turning the adjustable ring at the cap.
3. This model has exchangeable front covers/caps.
4. This model has a macro zoom function: working with 2 magnification rates while maintaining the same working distance: a larger field of view at a lower magnification and a zoom function for higher magnification.

## Generic models

### Intended use

Generic models can be used in general microscopy applications

## Hardware features

The wireless models do not have a microtouch and can be controlled via the software program on your connected device.

1. Your Dino-Lite has a microtouch function at the cable end of the device. Touching this sensor will capture the current image.
2. The wireless models do not have a microtouch and can be controlled via the software program on your connected device.
3. Models with the letter W in the article code have the macro zoom function: working with 2 magnification rates while maintaining the same working distance: a larger field of view at a lower magnification and a zoom function for higher magnification.

## Software installation

The DinoCapture and DinoXcope software is licensed from Anmo Electronics Corporation and is subject to an End User License Agreement (EULA) that users will have to accept during the installation process.

***Important notice: DO NOT connect the USB cable of the product to the PC before installing the software. Please check the separate software manuals that can be downloaded from [www.dino-lite.eu](http://www.dino-lite.eu).***

1. Download and install the DinoCapture and DinoXcope software and drivers. Alternatively, download and run the latest version of the software and drivers from the support/download section of the website: [www.dino-lite.eu](http://www.dino-lite.eu).
2. Click 'Next' and the Installshield wizard will start. (An 'Open File-security warning' may appear on some systems. Select Run' or 'YES'). Choose the language you want for the DinoCapture interface.
3. Read the User License Agreement. If you agree, press 'Yes' to continue or press 'no' to stop the installation.
4. Select a destination folder for the DinoCapture software. When done, press 'Next'. Click 'Install' to start installing the software. If the Windows security warning message appears, click 'Install this driver software anyway'.
5. When the installation is complete, selecting 'Finish' completes the software installation.
6. The DinoCapture software has an auto-update feature that will check for software updates when you start DinoCapture.
7. A full manual can be found in the help function of DinoCapture or as a pdf on the website: [www.dino-lite.eu](http://www.dino-lite.eu).

## Hardware installation

1. After full installation of the DinoCapture software and driver package, connect the device to one of the USB ports of your computer.
2. Please use a USB 2.0 port that is fully powered. Some USB ports on portable computers do not supply sufficient power.
3. The driver will be automatically installed. Please WAIT until the notification appears: 'Device driver software installed successfully'.
4. Now start DinoCapture by double clicking on the desktop icon.
5. The LED lights should go on and an image should appear in DinoCapture. If this is not the case, refer to the frequently asked-questions (FAQ) on [www.dino-lite.eu](http://www.dino-lite.eu).
6. For wireless models: for connection with the USB cable, follow the instructions above. For connection with the WiFi module: "After installing the USB connection adapter, follow the instructions 1-5 above. If you want to use the device with WiFi, connect the WiFi connection adapter. This can be used with a smartphone or with a Windows computer.

***Download and install DinoConnect from the Apple® App Store® or Google Play™. Or download and install DinoCapture for Windows, which can be downloaded from [www.dino-lite.eu](http://www.dino-lite.eu).***

### Hardware features

The wireless models do not have a microtouch and can be controlled via the software program on your connected device. At the center of the device, the adjustable dial is used to set the focus. The focus of the image depends on the distance to the object. Once you have focused on the object, you can read the magnification rate achieved from the number next to the  $\Delta$  symbol.

### Cleaning and maintenance

Avoid touching the lens with any substance. Clean the lens periodically with compressed air. Clean the body of the microscope regularly with a soft cloth dampened with disinfectant. Clean the detachable caps with a 50%-70% alcohol solution, do not use diethyl ether, 100% alcohol or alcohol gel.

***Dispose as electronic waste according to local regulations.***

### Warranty

The DinoCapture software is provided for use with a digital microscope supplied by AnMo Electronics or one of its distributors or resellers. The use of the DinoCapture software is guided by the End User License Agreement. The product is delivered with a two year warranty from the date of purchase by the end customer. Please note that accessories (i.e. detachable caps/extensions) are not covered under warranty. For warranty issues please contact the reseller or store where you bought the product.

### Support

If you have any problem or issue with Your Dino-Lite or the DinoCapture software, please contact your reseller or check the Dino-Lite Europe support pages at [www.dino-lite.eu](http://www.dino-lite.eu). An extensive user manual for DinoCapture can be found within the software and on the website: [www.dino-lite.eu](http://www.dino-lite.eu).

### IDCP BV

Manuscriptstraat 12-14, 1321NN Almere, The Netherlands  
Tel. +31 (0)20 6186322 | Fax +31 (0)20 6189692  
E-mail: [info@dino-lite.eu](mailto:info@dino-lite.eu) | Web: [www.dino-lite.eu](http://www.dino-lite.eu)

UKCA authorized representative; IDCP Scotland Limited, Suite 1F Ingram House, 227 Ingram Street, Glasgow G1 1DA

2024/Q1 AnMo/IDCP/Dino-Lite Europe.

No part of this publication may be reproduced or distributed unless for personal use in conjunction with usage of Dino-Lite digital microscopes.

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2024/Q1 © AnMo/IDCP
### **IDCP B.V./Dino-Lite Europe**